Pump In Style® Advanced

EN Instructions for use | SP Instrucciones de uso | FR Mode d'emploi

Backpack

On-the-go Tote

The Metro Bag™

Important Safeguards

When using electrical products, especially when children are present, basic safety precautions should always be followed.

READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

DANGER: To reduce the risk of electrocution:

- Always unplug electrical product immediately after use.
- Do not use while bathing.
- Do not place or store product where it can fall or be pulled into a tub or sink.
- Do not place or drop into water or other liquid.
- Do not reach for a product if it has fallen into water. Immediately unplug from electrical outlet.

WARNING: To avoid fire, electrocution, or serious burns:

- Do not leave product unattended when plugged into an electrical outlet.
- Close supervision is necessary when this product is used near children or persons with disabilities.
- Use the product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
- Keep cord away from heated surface.
- Never use while sleeping or when drowsy.
- Never drop or insert any object into any opening or tubing.
- Do not use outdoors, or operate where aerosol (spray) products are being used or where oxygen is being administered.
- Always inspect power adaptor and battery pack wires prior to use for damage or exposed wire. If damage is found, immediately discontinue use of power adaptor or battery pack and call Medela Customer Service at 1-800-435-8316.

The warning symbol identifies all instructions that are important to safety. Failure to observe these instructions can lead to injury or damage to the breastpump. When used in conjunction with the following words, the warning symbols stand for:

-  **WARNING** Can lead to serious injury or death.
-  **CAUTION** Can lead to minor injury.
-  **NOTE** Can lead to material damage.
-  **INFO** Useful or important information that is not related to safety.

WARNING: To avoid health risk and reduce the risk of injury:

- This product is intended for use by a single user only and should not be shared between users.
- Do not drive while pumping.
- Never use while pregnant, as pumping can induce labor.
- Clean and sanitize all parts that come in contact with your breast and breastmilk prior to first use.
- Wash all parts that come in contact with your breast and breastmilk after every use.
- Inspect all appropriate pump components before each use.
- Do NOT continue pumping for more than 2 consecutive pumping sessions if no results are achieved.
- Do not thaw frozen breastmilk in a microwave or in a pan of boiling water.
- If tubing becomes moldy, discontinue use and replace tubing.
- If breastmilk backup occurs you must clean tubing, faceplate and diaphragm before your next pumping session.

CAUTION: Can lead to minor injury:

- Do NOT wrap cord around adaptor body.
- Use only the power adaptor that comes with the Pump In Style® Advanced breastpump.
- Use only the Pump In Style Advanced Battery Pack that comes with the Pump In Style breastpump.
- Pump only with the breastpump in an upright position.
- Do NOT shorten tubing.
- Make sure the voltage of the power adaptor is compatible with the power source.
- Plug the power adaptor into the breastpump first and then into the wall socket.
- Do NOT use antibacterial or abrasive cleaners/detergents when cleaning breastpump or breastpump parts.
- Never put breastpump motor in water or a sterilizer, as you can cause permanent damage to the breastpump.
- Do not attempt to remove the breastshield from your breast while pumping. Turn the breastpump off and break the seal between your breast and breastshield with your finger, then remove breastshield from your breast.
- If pumping is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the breastshield with your finger and remove the breastshield from your breast.

SAVE THESE INSTRUCTIONS

Table of Contents

1. Cleaning Instructions	8-15
1.1 Before using for the first time	8
1.2 After each use.....	8-9
1.3 Sanitize daily	10-11
1.4 Tubing and faceplate cleaning.....	12-14
1.5 Cleaning your breastpump bag	15
2. Assembly	16-23
2.1 Assembly of pump kit	16-19
2.2 Correctly fitting breastshields	20
2.3 Power options	21-23
2.3.1 Power adaptor care	21
2.3.2 Battery Pack.....	22
2.3.3 Traveling outside of the U.S.	23
3. Pumping with Your PNSA	24-27
3.1 Device operation.....	24-26
4. Storing Breastmilk	28
5. Preparing & Feeding Breastmilk	29
5.1 Preparing breastmilk	29
5.2 Feeding breastmilk.....	29
6. Troubleshooting	30
7. Supplemental Information	31
8. Warranty	32

Take a moment to read through this entire instruction manual before using this product for the first time.

PLEASE SAVE THESE INSTRUCTIONS.

It is best to wait until your breastfeeding routine is established (approximately 4 weeks) before expressing breastmilk, unless otherwise advised by your healthcare professional.

Glossary of Terms

2-Phase Expression® technology – research based technology that mimics a baby’s natural nursing rhythm.

Expression Phase – slower sucking/pumping rhythm for gentle and efficient milk removal as quickly as possible.

Maximum Comfort Vacuum™ – highest vacuum setting where a mother still feel comfortable during pumping. It is different for every mother.

Stimulation Phase – fast sucking/pumping rhythm to stimulate the milk ejection reflex and to start the milk flowing.

Intended Use

For lactating women to express and collect milk from their breasts to complement breastfeeding.

If you have medical reasons or other needs for exclusively pumping, it is recommended that you use a hospital grade breastpump such as our Symphony breastpump with 2-Phase Expression technology. To learn more, visit us at www.medelabreastfeedingus.com.

Product Description

Pump In Style® Advanced (PNSA) is a personal use electric breastpump that includes 2-Phase Expression technology and is capable of single and double pumping.

Product Description

On-the-go Tote

Backpack



2x PersonalFit™ Breastshields M (24 mm)
(for other breastshield sizes see **Section 2.2**, page 20)
87073

2x connectors
87071

2x valves
87089

2x membranes
87088

2x bottles
6100050

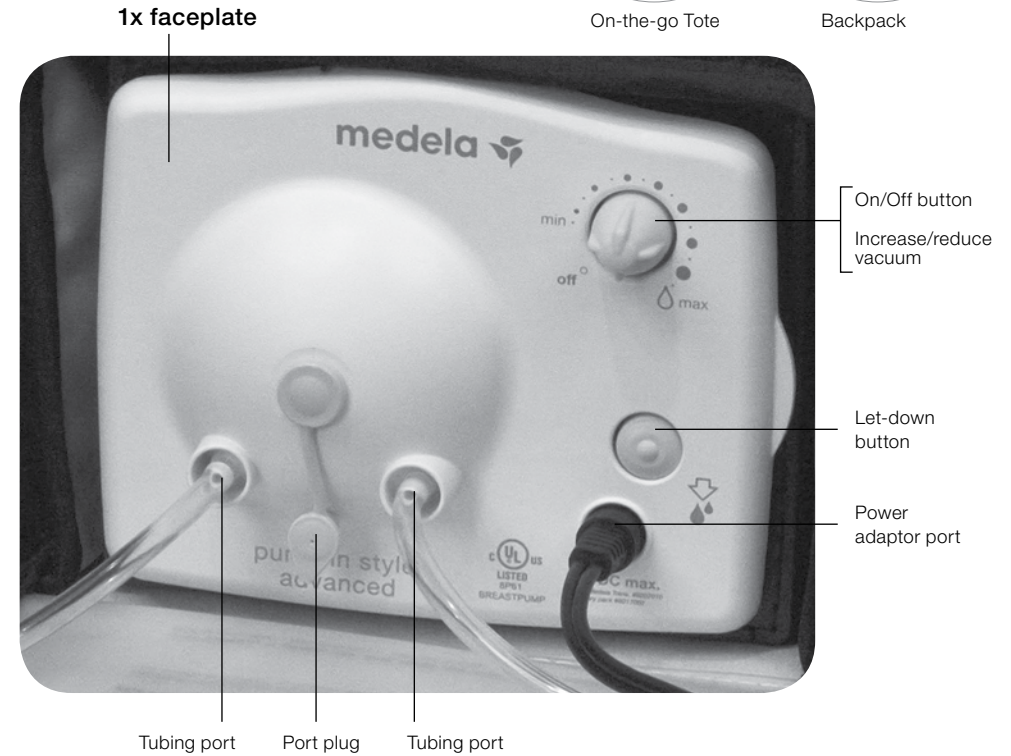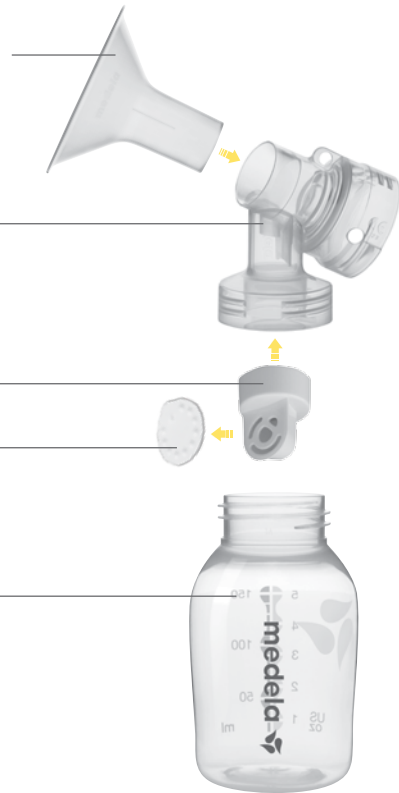2x lid
8107183

1x battery pack
Replacement 67553
Item 9017002

2x tubing
87212

1x power adaptor
Replacement 68030
Item 9207010

1x cooler bag (includes ice pack)
BCOOL001

1x ice pack
Item 8117010

Additional parts included with The Metro Bag™


Pump In Style®
Advanced
The Metro Bag

27 mm PersonalFit Breastshields..... 87274
Metro Bag including:
Flexible, Plastic Workspace 3007299
2 Mesh Storage Bags

1. Cleaning

1.1 Before using for the first time

Clean and sanitize the parts below before using your pump for the first time. Follow the cleaning and sanitizing instructions in **Sections 1.2 (start at step 4)** and **1.3** that follow.

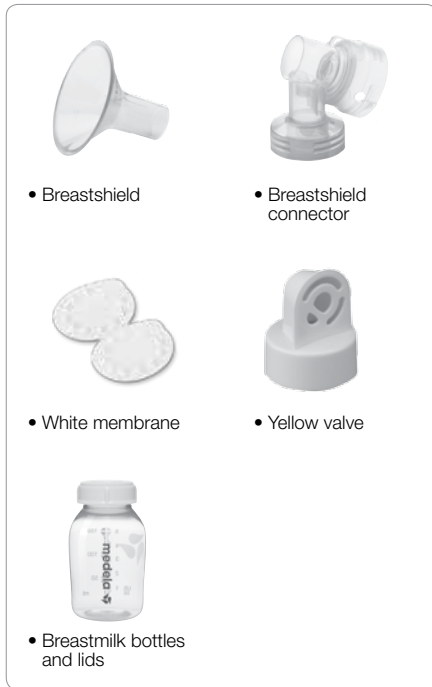Caution

- Only use drinking-quality tap or bottled water for cleaning.
- Take apart and wash all parts that come in contact with the breast and breastmilk immediately after use to avoid dry up of milk residues and to prevent growth of bacteria.

1.2 After each use

Clean all parts that come in contact with your breast and breastmilk. See image in **Section 1.1**.

Supplies Needed:

- Mild Dish Soap
- Clean Sink or Bowl
- Clean Dish Towel or Soft Brush
- Drinking Quality Water

Separate breastshields from breastshield connectors.

Twist and pull the yellow valves off the breastshield connectors.

Remove the white membranes from the yellow valves.

Washing Options:

Washing in sink:

Rinse in cool water all separated parts that came in contact with breast and breastmilk in order to remove breastmilk residue.

- Soak all separated parts in warm soapy water for 5 minutes.
- Clean with a clean dish-cloth or soft brush.
- Rinse all separated parts with clear water.
- Allow all parts to air dry in a clean area.
- Store dry parts when not in use.

Washing in dishwasher:

- Wash all separated parts on top rack of dishwasher.
- Allow all breastpump parts to air dry in a clean area.
- Store dry parts when not in use.

1. Cleaning (cont.)

Note

- **Wash** breastpump kit parts after each use.
- **Sanitize** breastpump kit parts once daily.

1.3 Sanitize daily

Wash hands thoroughly.

1

Refer to image in Section 1.1


Separate all parts that come in contact with your breast and breastmilk.

2

- Fill a pot with enough water to cover all parts.
- Bring water to a boil.
- Place parts in boiling water for 10 minutes.

3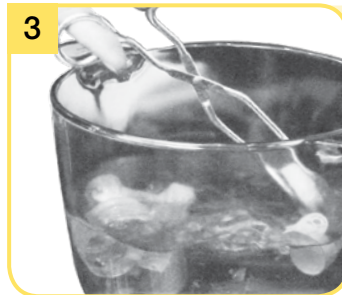

Allow water to cool and gently remove parts from water with tongs.

4

- Place parts on a clean surface and/or towel.
- Allow all parts to air dry.
- Store dry parts when not in use. Do NOT store wet or damp parts.

or

You may sanitize your kit parts by following the instructions on Medela's Quick Clean™ Micro-Steam™ bag.

1. Cleaning (cont.)

Caution

- Do not store wet or damp parts as mold may develop.
- If tubing becomes moldy, discontinue use and replace tubing. Contact Medela Customer Service at customer.service@medela.com or 1-800-435-8316.

Note


- Tubing should be washed if dirty or milk is present. Follow instructions in **Section 1.4**.

1.4 Tubing and faceplate cleaning


Clean the faceplate monthly. Also, inspect tubing after each pumping session. Clean tubing and faceplate if there is any sign of condensation and/or milk.

Turn off breastpump.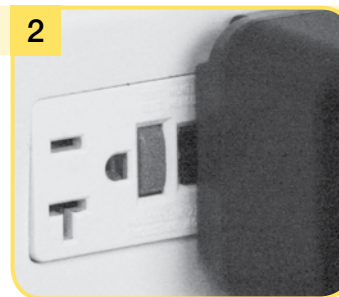

Unplug breastpump from power source.

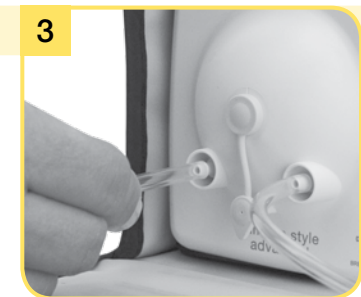
Remove tubing by pulling straight out of tubing ports. Do not wiggle or pull tubing at an angle.

Remove tubing from breastshield.

Rinse tubing in cool water to remove breastmilk.

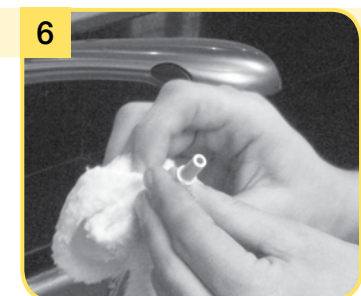- Wash tubing in warm soapy water.
- Rinse tubing with clear water.
- Hang to air dry.

Continued next page.

1. Cleaning (cont.)


Note

- Make sure faceplate and diaphragm are completely dry before reseating.
- The faceplate must be seated correctly and snapped together at all connection points. This ensures the pump will have adequate vacuum and optimum performance.

1.4 Tubing and faceplate cleaning (cont.)


Snap off the faceplate using the thumb tab.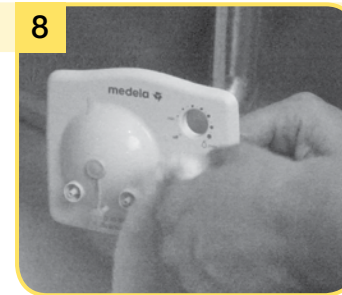

- Soak in warm soapy water for 5 minutes.
- Clean with a clean dish-cloth or soft brush
- Rinse parts with clear water.
- Allow to air dry in a clean area.

- Wipe diaphragm with a clean (no soap), damp cloth.
- Allow to air dry in a clean area.

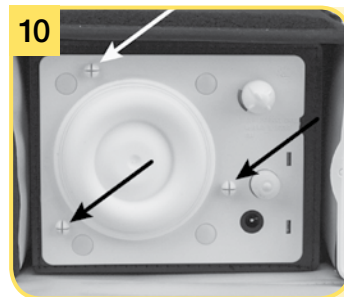Locate the 3 connection points on the reverse side of the faceplate and on the pump.

Align connection points and hold back of pump with one hand while pushing on the faceplate until it snaps into place.


1.5 Cleaning your breastpump bag

Use a damp (not wet) cloth to wipe the breastpump bag after each use.

2. Assembly

Parts you will need for this step: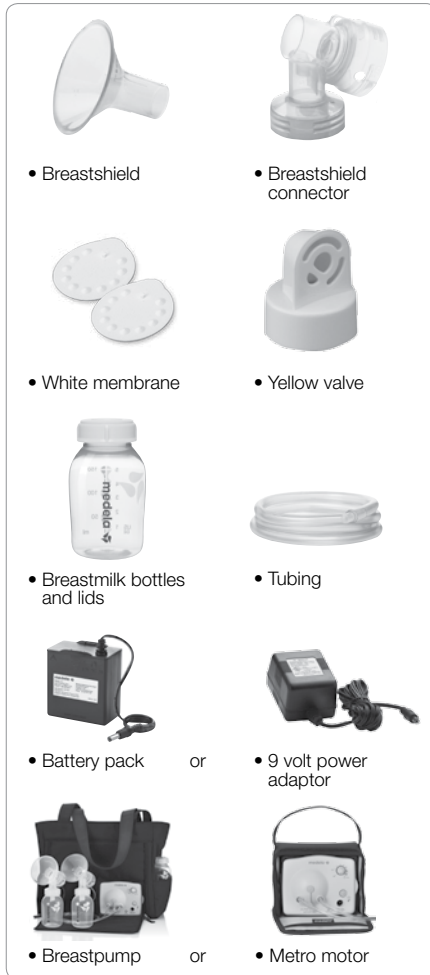

! Note

- Inspect valves for cracks or chips and inspect membranes for holes, tears or warping. If you notice any damage to these parts, replace them immediately.

2.1 Assembly of pump kit

1 Wash hands before touching breastpump, kit and breasts and avoid touching the inside of containers or lids.

2 Place your breastpump on a clean and dry surface. Ensure that the surface is solid and secure so your breastpump will not tip over or fall.

3 Securely attach breastshield to breastshield connector.

4 Snap membranes onto valves until membranes lie completely flat on valve.

5 Push assembled yellow valves and white membranes onto bottom of breastshield connectors.

6 Screw bottles onto breastshield connectors.

Continued next page.

2. Assembly (cont.)


Warning

- Use only the power adaptor supplied with the Pump In Style Advanced.
- Always inspect the 9 volt power adaptor and Battery Pack cords prior to use for damaged, kinked, twisted, frayed or exposed wires. If damaged, immediately discontinue use and call Medela Customer Service at 1-800-435-8316.

Twists, kinks or bends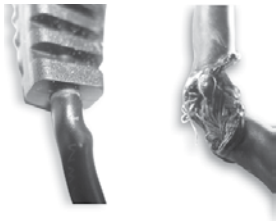

Has bulges Exposed wires

Note

- See **Section 2.3.2** Battery Pack Assembly for instructions on the use of the Battery Pack.
- See **Section 2.3.3** Traveling Outside the U.S. for information on using your device internationally.

2.1 Assembly of pump kit (cont.)

Insert hard yellow adapter ends of tubing into hole on the back of breastshield connectors.

Double pumping: Insert free ends of tubing onto both ports on front of breastpump faceplate.

Single pumping: Insert free end of tubing onto one port on front of breastpump faceplate. Place port cap over the unused port.

Plug the metal fitting on end of 9 volt power adaptor or Battery Pack cord into the power jack on front of breastpump faceplate.

Plug 9 volt power adaptor into a standard electrical outlet.

Refer to Power Adaptor Care in Section 2.2.1

2. Assembly (cont.)

Caution

- While some discomfort may be felt when first using a breastpump, using a breastpump should not cause pain. If you are unsure about breastshield sizing, please contact a healthcare professional or breastfeeding specialist who can help you get a proper fit.

2.2 Correctly fitting breastshields

Properly fitting breastshields will ensure comfort and efficient milk expression. To determine if you have the correct size, look at the following diagram:

- The nipple should be centered in the tunnel of the shield.
- The nipple should move freely without rubbing when the pump is turned on.
- Minimal or no part of your areola should be pulled into the tunnel of the breastshield.
- You should see a gentle, rhythmic motion in the breast with each cycle of the pump.
- After pumping, your breast should feel much less full with no areas of hardness.

Correct fit

Medela offers smaller and larger sizes which are available at many local retailers or visit www.shopmedela.com.

PersonalFit Breastshields

21 mm (Small)

Item 87072

24 mm (Medium)

Item 87073

27 mm (Large)

Item 87274

30 mm (X-Large)

Item 87075

36 mm (XX-Large)

Item 87084

Choosing the right size PersonalFit™ breastshield:

1. Determine the size you're currently using. If you're not sure, look for the size embossed on your breastshield (see picture). 24 mm (M) breastshields are provided with Medela breastpumps.

2. While pumping, compare your fit to the images below as a sizing guideline.

Correct fit

Your breastshield is **too small**;
try a larger size

Your breastshield is **too large**;
try a smaller size

2.3 Power options

2.3.1 Power adaptor care


To care for your adaptor be sure to follow these storage directions.

Do not wrap cord of power adaptor around the plug body.

Correct!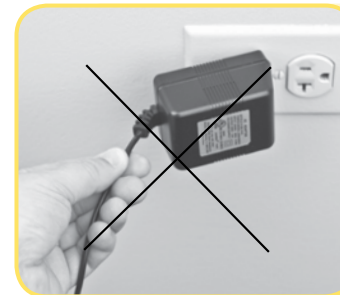

Do not unplug power adaptor by pulling on the cord.

Correct!

2. Assembly (cont.)


Note

- Remove the batteries if the Battery Pack is not going to be used for an extended period of time.
- If breastpump does not turn on, make sure the batteries have been inserted properly. If this does not solve the problem refer to the troubleshooting section of this instruction booklet or contact Medela Customer Service at 1-800-435-8316.
- Each set of batteries will provide approximately 2 hours of pumping time.

2.3 Power options (cont.)

2.3.2 Battery Pack

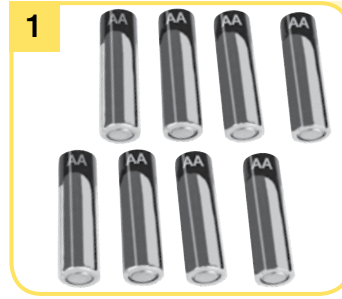Use 8 AA rechargeable batteries or alkaline batteries.

Replace all batteries at the same time.

Slide off both battery covers. Four (4) batteries are placed on each side of battery pack.

Place the NEGATIVE (-) end of the battery against the front of the spring by pushing the battery in and down in one motion until the battery snaps into place.

The POSITIVE (+) end of the battery should be near the positive sign on the battery pack.

Replace battery cover.

2.3.3 Traveling outside the U.S.

When traveling outside of the U.S., there are 2 power options:

Use the Battery Pack that was included with your Pump In Style® Advanced breastpump.

or

Purchase the 9 volt Vehicle Lighter Adaptor (sold separately). Locate a Medela Retailer at: <http://medela.findlocation.com> or visit our online store at www.shopmedela.com.

3. Pumping

Caution

- Always inspect breastshield, breastshield connectors, white membranes, yellow valves, and tubing prior to use for cleanliness. If dirty, see **Section 1.2**.
- If you experience pain or other symptoms while pumping seek advice from your healthcare professional or breastfeeding specialist. See **Section 7** for more information.
- Do not try and express with vacuum that is too high and uncomfortable (painful). The pain, along with potential breast and nipple trauma may decrease milk output.
- Make sure tubing is not kinked or pinched while pumping.


Info

- Research has shown that you will pump more efficiently – get more milk in less time – when pumping at Maximum Comfort Vacuum™ during the Expression Phase.
- You should reassess your Maximum Comfort Vacuum throughout your pumping experience because it can change during the different stages of lactation.
- Do not tilt or overfill containers when pumping.

3.1 Device operation

1 Wash hands before touching breastpump, kit and breasts and avoid touching the inside of containers or lids.

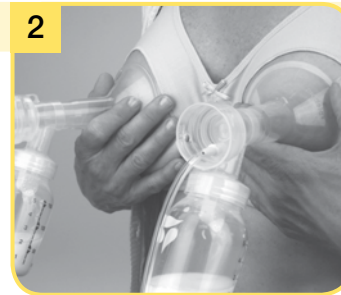2 Center the assembled breastshields over your nipples.

3 To begin pumping, turn dial clockwise from the off position. Your pumping session will start in the Stimulation Phase. While pumping, adjust the speed/vacuum by turning the dial to your comfort level.

4 After two minutes, the pump will automatically change to the Expression Phase.

- a. If you let-down (your milk starts to flow) sooner than 2 minutes, press the let-down  button.
- b. If let-down has not happened during the Stimulation Phase and the pump has switched to the Expression Phase, press the let-down  button once to return to the Stimulation Phase.

5 To find your Maximum Comfort Vacuum, increase speed/vacuum until pumping feels slightly uncomfortable (not painful), then decrease slightly.

6 When your pumping session is over unplug tubing from the back of the breastshield(s) before setting down the bottles.

Continued next page.


3. Pumping (cont.)

Info

- **How often should you pump?** A breastpump is a replacement for when you are separated from your baby. It is important to pump when the baby would be breastfeeding, which usually means a working mother pumps 2-3 times during an 8 hour working day.
- **How long should your pumping session last?** Pumping times can vary from mother to mother sometimes 15 minutes, sometimes up to 30 minutes.
- **How should your breasts feel after pumping?** After pumping, your breasts should feel soft and there should be no firm areas. Before pumping, your breasts will have a firm, heavy feeling. After pumping make sure your breasts are not firm or have lumpy areas. This could indicate that the breast is not draining all over. See **Section 7** for more information.


3.1 Device operation (cont.)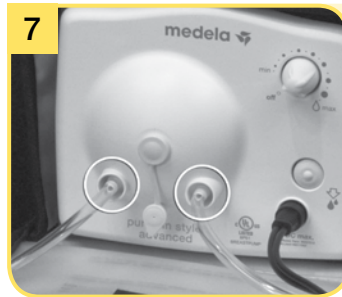


7 Allow the pump to continue to run while you store your breastmilk. This will allow any condensation that has formed due to the natural results of humidity to dry in the tubing.

8 Turn off breastpump.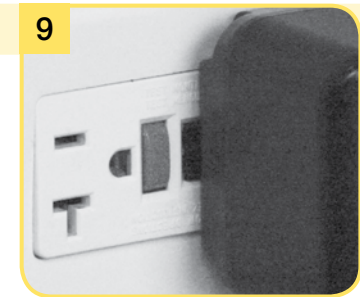

9 Unplug breastpump from power source.

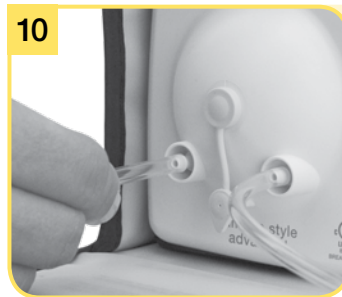10 Prior to storing your pump remove tubing by pulling straight out of tubing ports. Do not wiggle or pull tubing at an angle.

11 Disassemble and clean per Cleaning Instructions in **Section 1**.

4. Storing Breastmilk

Caution

- Check with your hospital or healthcare provider if you require specific storage instructions.
- Do not thaw frozen breastmilk in a microwave or in a pan of boiling water.

Warning

- Do not microwave breastmilk. Microwaving can cause severe burns to baby's mouth from hot spots that develop in the milk during microwaving. (Microwaving can also change the composition of breastmilk.)

Note

- If the ice pack is frozen solid, it will safely cool your breastmilk for up to 12 hours in a room temperature environment.

4.1 Storing breastmilk in Pump In Style® Advanced cooler bag

The Pump In Style Advanced cooler bag requires one contoured ice pack (#87092, included) for cooling up to four 5 oz bottles of breastmilk.

- As soon as you are finished pumping, place the breastmilk bottle or breastmilk storage bag in the cooler bag with the ice pack.
- Transfer breastmilk bottles or breastmilk storage bag to the refrigerator or freezer once you are home.

5. Preparing and Feeding Breastmilk

Freshly Expressed Breastmilk Storage Guidelines (For Healthy Term Babies)

Room Temperature	Cooler with Ice Pack	Refrigerator	Freezer	Thawed Breastmilk
4–6 hours at 66–78 °F (19–26 °C)	24 hours at 59 °F (15 °C)	3–8 days at 39 °F or lower (4 °C)	6–12 months 0–4 °F (-18–-20 °C)	use within 24 hrs

5.1 Preparing breastmilk

- Thaw breastmilk overnight in the refrigerator. Thawed breastmilk is safe in the refrigerator for 24 hours. Do not refreeze thawed breastmilk.
- Quickly thaw breastmilk by holding the bottle under warm running water.
- Place the sealed bottle in a bowl of warm water for 20 minutes to bring it to body temperature.
- If you are adding expressed breastmilk to a container of already frozen breastmilk, make sure to add a lesser amount than the already frozen amount.

5.2 Feeding breastmilk

It is recommended that breastfeeding is well established prior to bottle feeding your baby.

- Always inspect the bottle, nipple and other components immediately before and after each use. If nipple appears cracked or torn, discontinue use immediately.
- To prevent possible choking hazard, test strength of nipple by pulling on bulb portion of the nipple.
- Do not attempt to enlarge the nipple hole.
- Infants must not be bottle fed without adult supervision.
- Nipple should not be used as a pacifier.

6. Troubleshooting

Low or No Suction	<p>Valve and Membrane Check</p> <ul style="list-style-type: none"> Remove and separate valve from membrane Inspect the valve for chips or cracks. Inspect the membrane for holes or tears. If flawed or damaged replace valve and/or membrane prior to pumping Wash valve and membrane per Cleaning section Reassemble. Make sure the membrane lays flat on the valve, and if it does not, replace the valve and/or membrane prior to pumping <p>Breastshield and Breastshield Connector Check</p> <ul style="list-style-type: none"> Inspect breastshield and breastshield connector to make sure there are no cracks and they securely attach to each other <p>Faceplate Check</p> <ul style="list-style-type: none"> Wash faceplate and backplate per Milk Overflow section steps 1.4 on pages 12-15. Reassemble faceplate and backplate, and reattach tubing to faceplate port <p>Connection Point Check</p> <p>Inspect following connection points to ensure attachment is secure</p> <ul style="list-style-type: none"> Tubing adapter to breastshield connector Tubing end to faceplate port Faceplate to backplate Test vacuum. If suction is not improved, contact Medela Customer Service at 1-800-435-8316.
Audible air coming from port cap/faceplate	<ul style="list-style-type: none"> There is a built-in air leak for vacuum control, some audible air is normal. <p>⚠ CAUTION: Do not attempt to plug hole in cap or port with additional material.</p>
Power source	<ul style="list-style-type: none"> If pump is not turning on with the power adaptor try the battery pack. If the pump is not turning on with the battery pack, be sure the batteries have been inserted correctly. If pump is still not working with the power adaptor or the battery pack, contact Medela Customer Service at 1-800-435-8316.
Faceplate port broken/ cracked	<ul style="list-style-type: none"> Contact Medela Customer Service at 1-800-435-8316 or order online at www.shopmedela.com. <p>⚠ CAUTION: If you are feeling a loss of suction with the broken faceplate, do not use the pump until the new faceplate arrives.</p>
Condensation or milk in tubing	<ul style="list-style-type: none"> Refer to "Tubing and faceplate cleaning" section 1.4.
No "let-down" or breastmilk expression	<ul style="list-style-type: none"> Ensure your breastpump is assembled correctly and that there is suction. Relax and take a 10-15 minute break if let down is not achieved after 2 consecutive pumping sessions. Consult with your healthcare provider or breastfeeding specialist if expression does not occur.

If you have not resolved the problem with your breastpump or you have further questions, please contact Medela Customer Service at 1-800-435-8316 or email customer.service@medela.com.

7. Supplemental Information

Following are some common breastfeeding related conditions. If you experience any of these symptoms, contact a healthcare professional or breastfeeding specialist.

	Symptom	Potential Causes
Engorgement	Breasts are hard and uncomfortable, possibly with reddened areas.	Milk not draining properly from breast. Milk can drain back into the tissue causing areas to swell and become tender.
Blocked/Plugged Ducts	Area of breast will look reddened and may be tender to touch.	Milk not being drained from a specific duct. The area becomes 'clogged' up and milk is then prevented from flowing.
Mastitis	It often follows on from engorgement but can just occur suddenly. The initial symptoms are similar to flu – fatigue, headache and muscle aches, fever and localized breast tenderness. It can be in one or even both breasts and needs immediate treatment.	A bacterial infection in the breast tissue which is often accompanied by cracked nipples.
<p>A Lactation Professional or Healthcare Professional can give you guidance in breastfeeding your baby and using a breastpump. Their guidance is valuable in terms of the lifetime of health benefits for both you and your baby. A listing of Lactation Professionals are available at www.medelalocator.com or call 1-800-TELL-YOU (1-800-835-5968).</p>		

8. Warranty

Caution

This is a single user product. Use by more than one person may present a health risk and performance questions and voids the warranty.

This product is warranted by Medela, Inc. to the original retail purchaser to be free from defects in material and workmanship for the period of one year for pump mechanism (90 days for parts and accessories) from the date of purchase. In the event of a defect, Medela will repair or, at Medela's option, replace this product, without charge for such replacement, parts or labor. Purchaser shall bear all expense for returning this product to Medela. This warranty does not apply to any product used commercially or which has been subjected to misuse, abuse or alteration.

ANY AND ALL IMPLIED WARRANTIES, INCLUDING THE WARRANTY OF MERCHANTABILITY, ARE LIMITED TO A DURATION OF 90 DAYS FROM DATE OF PURCHASE. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE.

Before you make a claim under this warranty, it may save time and expense to call Medela Customer Service (toll free) at 1-800-435-8316. You may also call this number for additional information concerning this warranty. If you wish to make a claim under this warranty, you must return this product to Medela with a return authorization number received from Medela Customer Service, prepaid, together with your dated bill of sale or other proof of purchase and a brief statement of the problem to the following address:

Medela, Inc.
4501 Prime Parkway
McHenry, IL 60050
ATTENTION: RETURNS

Call first for authorization number. Returns not accepted without an authorization number.

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Printed in the USA. Medela, 2-Phase Expression and Pump In Style are registered trademarks of Medela Holding, AG.
Quick Clean, Micro-Steam, Maximum Comfort Vacuum, The Metro Bag and PersonalFit are trademarks of Medela, Inc.
Medela Canada, Inc.,
Quick Clean, Micro-Steam, Maximum Comfort Vacuum, The Metro Bag y PersonalFit son marcas comerciales de Medela, Inc.