


Skin Barrier Arc

Underneath the Appliance:

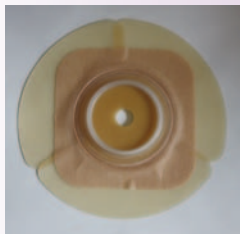

The Skin Barrier Arcs can be placed directly on the skin around the stoma to create a smooth platform for your pouch.

The Skin Barrier Arcs protect the skin from trauma caused by repetitive removal of tapes and adhesives. You can apply your wafer or tapes directly over the Skin Barrier Arcs.

Instructions for use:


1. Prepare the skin and make sure it is clean and dry, using a Peri-Stoma Cleanser & Adhesive Remover wipe.
2. We suggest using a skin barrier wipe or spray to protect your skin and let it completely dry.
3. Remove the white paper backing from the skin barrier arc.
4. Place the skin barrier arc directly on the skin around the stoma.
5. Use your fingertips to smooth and press the skin barrier arc onto the skin so the skin barrier arc and skin form a secure seal.
6. Apply the ostomy appliance.



Item# SNS20630 - 1" Wide - 30 Strips/Tray

Item# SNS21120 - 2" X-tra Wide - 20 Strips/Tray

Over the Appliance:

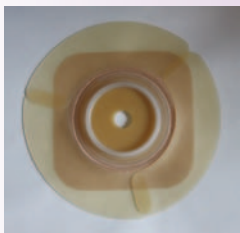

Skin Barrier Arcs can be placed around the wafer to secure the edges from lifting up, creasing or loosening. Picture frame around the edges of the wafer using 1-3 arcs.

Skin Barrier Arcs provide an excellent skin friendly alternative to waterproof tapes to give you a waterproof seal around the wafer when bathing or showering.

Instructions for use:

1. Prepare the skin and make sure it is clean and dry, using a Peri-Stoma Cleanser & Adhesive Remover wipe.
2. Remove the white paper backing from the skin barrier arc.
3. Place the skin barrier arc directly around the edges of the wafer or appliance forming a picture frame around the border of the wafer. (see picture) Use 1-3 arcs.
4. Use your fingertips to smooth and press the skin barrier arc around the appliance or wafer to create a secure waterproof seal.